

Bienvenue sur


moovcare®

PATIENT AND CAREGIVER USER GUIDE

The Moovcare® Toxicity app is a medical device. In case of emergency, please contact your doctor or the emergency room.

Version 1.0.1

Notice publication: May 2026

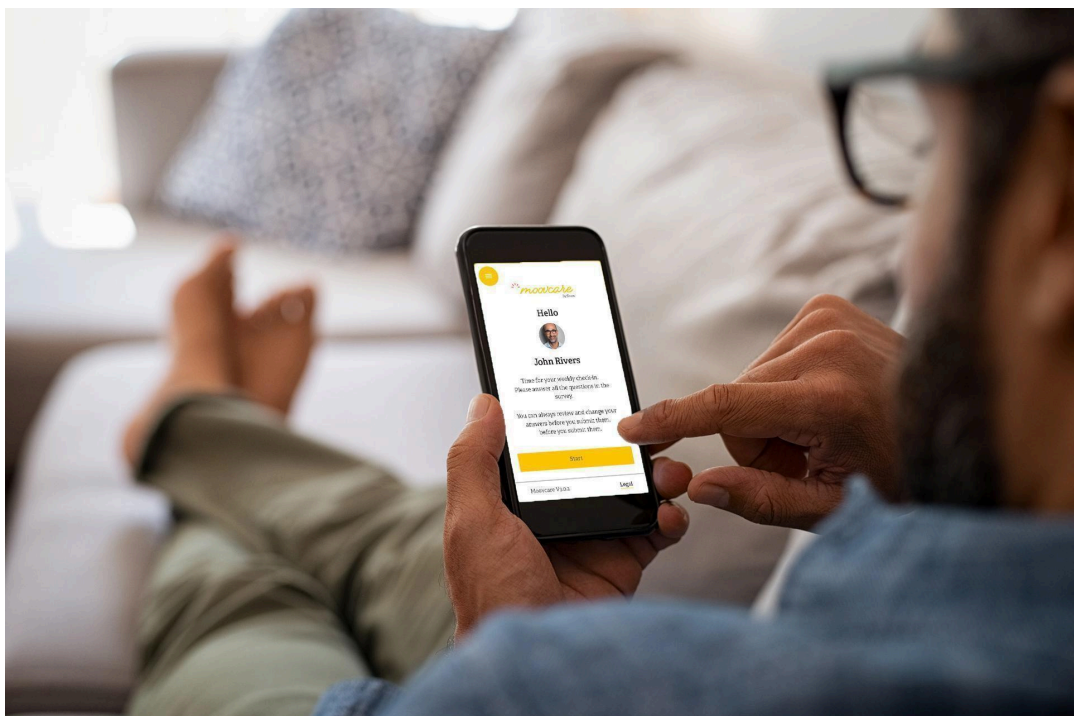

	Manufacturer:	SIVAN INNOVATION Ltd. Hatidhar 5 4366507 London, Israel
	Representative:	KAPSIKUM 4 rue René Razel 91400, Saclay, France
	Importer:	Sivan France 163 Quai Aulagnier F-92600 Asnières-sur-Seine, France

***Please read this leaflet carefully before using
MOOV CARE® Toxicity.***

- We advise you to keep this guide in case you need to review the web application's usage process. If you wish to access previous usage instructions or receive a printed copy, please send your request to: **contact@moovcare.com**.
- For any questions, advice or information, please contact your doctor.
- This medical device has been prescribed specifically for you. Do not share your access codes with anyone else, even if their symptoms are similar to yours, as this could compromise the proper functioning of this device for your medical monitoring.
- Technical support contact details:
support@moovcare.com: Response within 24 hours.

Service d'assistance - patients : 09 72 66 22 11 (free call)

Support hours: Monday to Friday, 9:00 AM to 12:00 PM and 2:00 PM to 5:00 PM
(except public holidays)

Sommaire



1. Operating principle.....	4
1.1 In practice.....	4
1.2 Professional users of Moovcare®.....	4
2. Instructions for use.....	5
2.1 Before we begin.....	5
2.2 Description of patient characteristics.....	6
2.2.1 Registration to benefit from the Moovcare® Toxicity application.....	6
2.2.2. Your questionnaire.....	10
2.2.3 How do I change my settings?.....	12
2.2.4. View data history.....	13
2.2.5. Do you wish to stop the Moovcare® monitoring?.....	14
2.2.6. Documentation/aide.....	14
2.2.7 Ask for help from a relative or caregiver.....	14
3. Who will be eligible for Moovcare? Toxicity?.....	15
4. Warnings.....	15
6. Personal data.....	15
7. Symbols.....	17
8. Date of manufacture.....	18
9. Legal notice.....	18


1. Operating principle

MOOV CARE® Toxicity is a medical device designed to monitor adult patients with metastatic cancer undergoing chemotherapy, targeted therapies, and/or immunotherapy in order to detect any toxicity related to cancer treatments. It is a software program based on an algorithm that analyzes clinical symptoms. These symptoms, which can be precursors to complications, are collected using questionnaires completed by the patients.

It allows for active, individual and remote monitoring of patients by their doctor.

1.1 In practice

Each week, you will receive an email with a link to a questionnaire about your symptoms. You can complete this form on your phone, tablet, or computer. If Moovcare® detects an anomaly, your care team will be notified immediately.

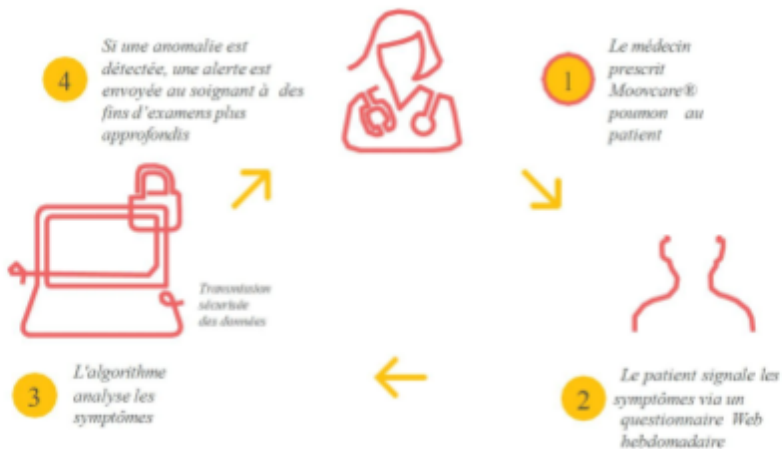

1.2 Professional users of Moovcare®

Referring physician

Your referring physician is the doctor who prescribed the Moovcare® monitoring application. All your answers are available on the Moovcare® dashboard Toxicity from the doctor in order to continuously monitor your health. If an anomaly is detected, an alert is sent immediately.

The other doctors at the care center

In case of absence, the referring physician may designate a secondary physician to monitor your symptoms.

The medical staff and the doctor's associates

Staff members designated by doctors may access your data to facilitate your follow-up.

L'application Moovcare® Toxicity is designed to remotely monitor toxicities related to cancer treatments, in order to detect any complications early.

2. Instructions for use

2.1 Before we begin

Moovcare® Toxicity is a medical monitoring device.

The app does not replace a doctor's diagnosis but functions as a digital support tool for remotely monitoring the symptoms of patients with metastatic cancer undergoing treatment. Moovcare® allows you to enter your symptoms, which will then be immediately transmitted to your doctor.

Note : At the end of your questionnaire, you have a free text field to allow you to provide further medical information or report new symptoms.

Personalized tracking

Moovcare® provides personalized monitoring and ongoing communication with your referring physician via a weekly questionnaire. This questionnaire is analyzed by Moovcare® Toxicity.

No installation required

Moovcare® Toxicity This is a web application; no download or installation is required. Internet access and an email address are mandatory. We recommend using a Gmail account to receive Moovcare® Toxicity emails and notifications.

Moovcare® Toxicity is compatible with the following versions of search engines:

- Chrome 141
- Firefox 141
- Internet Explorer 11
- Safari 18
- Edge 144

And also with the following operators:

- iOS 26
- Android 16
- Windows 11
- OSX 26

Data security

Moovcare® Toxicity uses the highest level of security to ensure the confidentiality of your health data.

All databases linked to the system comply with national data protection regulations. The data centers are located in France within a certified company.

Symptoms Questionnaire

At a pre-defined pace with the medical team, You will receive an email with a link to a simple questionnaire allowing you to report your symptoms on your phone, tablet, or computer. You can also use Moovcare® Toxicity to report symptoms whenever you want.

What kind of questions will I be asked?

Moovcare® Toxicity will ask you to report typical symptoms and to indicate the severity, if applicable. This may be a yes/no question, or an assessment of a symptom from minor to major. Your referring physician will be notified if an abnormality is detected.

2.2 Description of patient characteristics

2.2.1 Registration to benefit from the Moovcare® Toxicity application

Step 1: Tracking with the Moovcare® app

Moovcare® Toxicity is a class IIa medical device

Step 2: Eligibility Criteria

To determine your eligibility for Moovcare® Toxicity monitoring, your doctor will verify that you meet the eligibility criteria:

- You are over 16 years old.
- You are currently undergoing treatment with chemotherapy, targeted therapy and/or immunotherapy

Note: When you register in the Moovcare® Toxicity system, the doctor records your medical information about your current condition.

You can choose a family member or trusted person (such as a home caregiver) to help you complete the Moovcare® Toxicity Questionnaire at any time. However, this person will not have direct access to your questionnaire.

If you deem it necessary, please inform your referring physician.

Step 3: Activate your account

Once registered on Moovcare® Toxicity by your referring physician (or an accredited member of the care team), **You will receive an email to activate your account.**

- If you don't see the email in your inbox, check your junk mail or spam folder. Once you find the email, mark it as saved.
- "legitimate" so that subsequent emails arrive in your inbox.
- If you cannot find the activation email in your spam folder, it is likely that the email address you entered during registration is incorrect. Please contact technical support.

And then what?



You will receive an email to confirm the activation of your account, titled "Welcome to Moovcare® Toxicity." This email will give you access refer to the Moovcare® Toxicity system documentation and begin filling out your first questionnaire

You will be asked to confirm your email address. Click Confirm to open the Moovcare® Toxicity app. You will then be asked to create a password of at least 12 characters, including numbers and letters. Special characters are allowed. Once you have done this, click Set Password. Your password cannot contain your first or last name, nor any part of your email address.

The color bar works like this: **rouge/ orange:** Invalid


Green: valid

If the color is orange/red, the "set Password" bar will not be displayed.

Note : You will be asked for your password each time you wish to log in.


8 to the application. You can save it to your device so that your browser recognizes it.

3

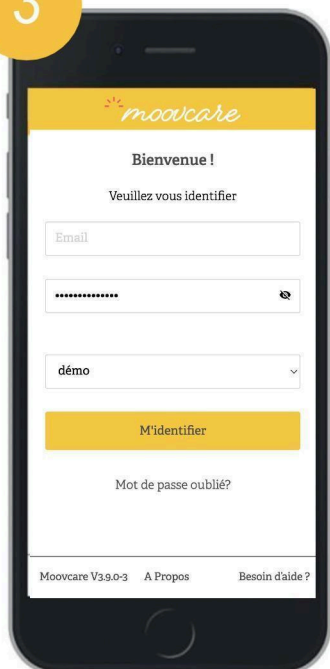

The login screen features the Moovcare logo at the top. Below it, the text 'Bienvenue !' is followed by 'Veuillez vous identifier'. There are three input fields: 'Email', a password field with a toggle icon, and a dropdown menu currently showing 'd mo'. A yellow 'M'identifier' button is below these fields. A link 'Mot de passe oubli ?' is at the bottom. The footer contains 'Moovcare V3.9.0-3', 'A Propos', and 'Besoin d'aide ?'.


4

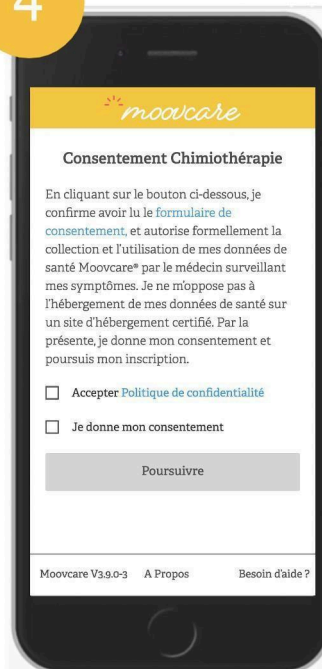

The consent screen features the Moovcare logo at the top. The title is 'Consentement Chimioth rapie'. The text explains that clicking the button confirms agreement to the collection and use of health data. It includes a link to the 'formulaire de consentement'. Below the text are two checkboxes: 'Accepter Politique de confidentialit ' and 'Je donne mon consentement'. A grey 'Poursuivre' button is at the bottom. The footer contains 'Moovcare V3.9.0-3', 'A Propos', and 'Besoin d'aide ?'.

The login screen will appear. Enter your password and click Log in.

The consent screen will open. Read it carefully; it concerns your agreement to the collection and use of your health data in Moovcare . Check the box I donate I agree and then click Continue.

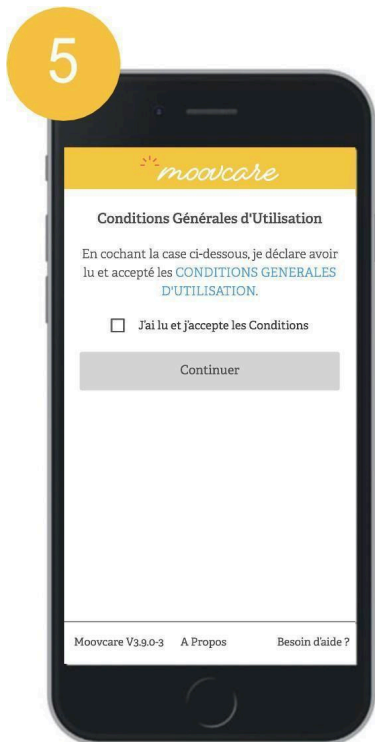


Next, the Terms of Use will appear. Please read them carefully.



You will then receive a text message on your phone with a code to identify yourself. Return to the Moovcare® app and enter the authentication code, then click Confirm.

Note : If there is an error in your phone number (you can check the one used to send the SMS on this same screen), click on **Change phone number** to correct it and receive a new code.

2.2.2. Your questionnaire

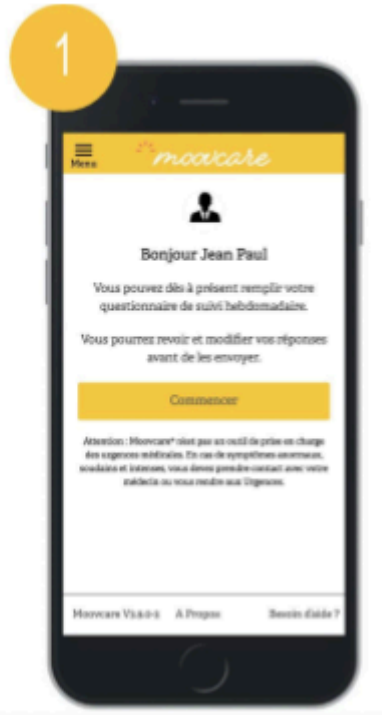

Once your account is activated, you will be able to click on Start to fill out your first questionnaire. At a predetermined pace, you will then receive a link to your Moovcare® space to complete your questionnaire.

By clicking Start, you will be directed to the first question. You will be asked to report any characteristic symptoms you may be experiencing.

The image shows a smartphone screen with the Moovcare app interface. At the top, there is a yellow header with the Moovcare logo and a close button (X). Below the header, the user's name 'Jean Paul' is displayed. The main content area contains instructions in French: 'Faîtes défiler vos réponses pour les relire, ensuite cliquez sur Envoyer' and 'Vous avez la possibilité de modifier vos réponses en cliquant sur la case de la question correspondante.' Below this is a yellow button labeled 'Envoyer'. There are two question boxes visible. The first question is 'Au cours des 7 derniers jours, avez-vous diminué votre consommation d'aliments et de boissons ?' with the answer 'Un petit peu' and a pencil icon for editing. The second question is 'Au cours des 7 derniers jours, à quelle fréquence avez-vous ressenti une douleur ?'. At the bottom of the screen, there is a footer with 'Moovcare V3.9.0-3' and 'Besoin d'aide ?'.

Note

The last question gives you access to a free text field. This is strictly reserved for medical information such as a change in your health status or new symptoms not specified in the questionnaire. Check your answers, then click Submit. You will not be able to change your answers. After this screen, you can close the page and you will receive a confirmation email shortly.



You can also use the Moovcare® questionnaire to report symptoms that may appear at any time between questionnaires, or request a new questionnaire using this address: <https://patient.moovcare.com/login/fr>

Vous avez oublié ou tardé à répondre au questionnaire ? Si vous ne répondez pas au questionnaire, votre équipe médicale en est informée.

Cependant, nous vous conseillons de suivre autant que possible les recommandations.


Une fois votre questionnaire envoyé, nous vous invitons à vérifier que vous avez bien reçu la confirmation d'envoi par email. Sinon, nous vous recommandons de soumettre un nouveau questionnaire ou d'appeler le support technique Moovcare.

2.2.3 How do I change my settings?

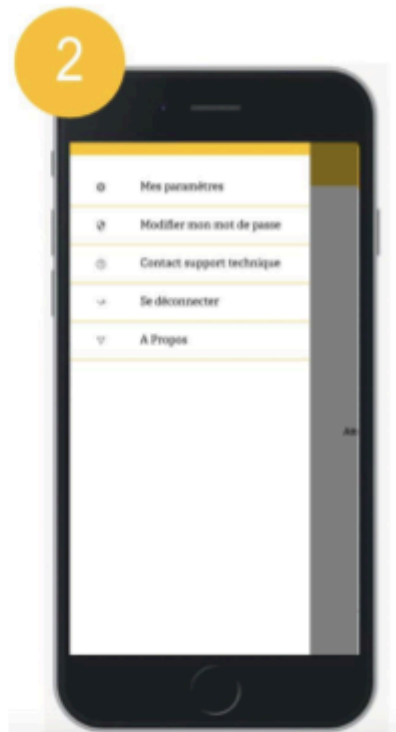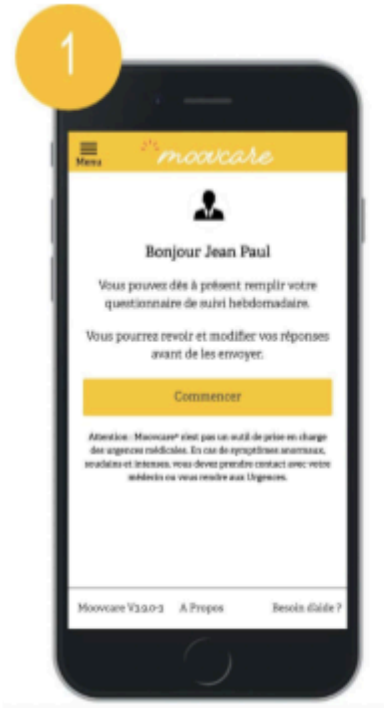

You can directly manage and modify certain data concerning you.
On the homepage, click on the menu in the top left corner.

If you wish, you can change your password from this menu.

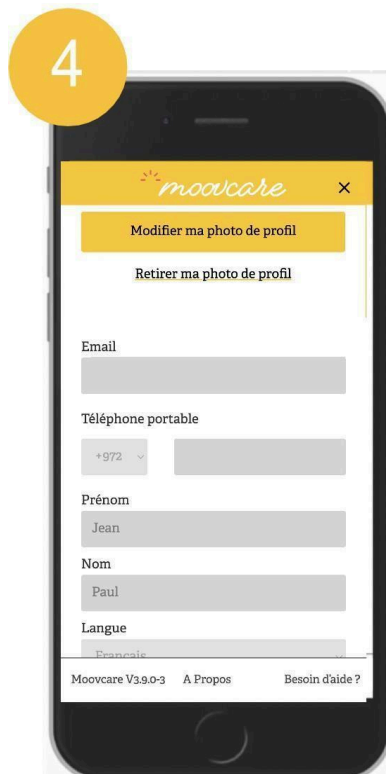

In My settings, you can change the questionnaire language.

2.2.4. View data history

You have the right to access your medical data. Your referring physician can export your questionnaire history if necessary.

2.2.5. Do you wish to stop the Moovcare® monitoring?


If you no longer wish to be monitored by Moovcare®, you can withdraw your consent. On the Welcome page, select My settings from the menu in the top left corner. At the bottom of the list, you will see Withdraw my consent for Moovcare® Toxicity.

By clicking on this option, a screen will appear to confirm that you wish to stop the Moovcare® Toxicity monitoring.

If you wish to be monitored by Moovcare® again

If you experience any toxicity, contact your primary care physician. They will send you an email to help you reconnect.

using your email address and the same password you used previously. You will then need to give your consent again.

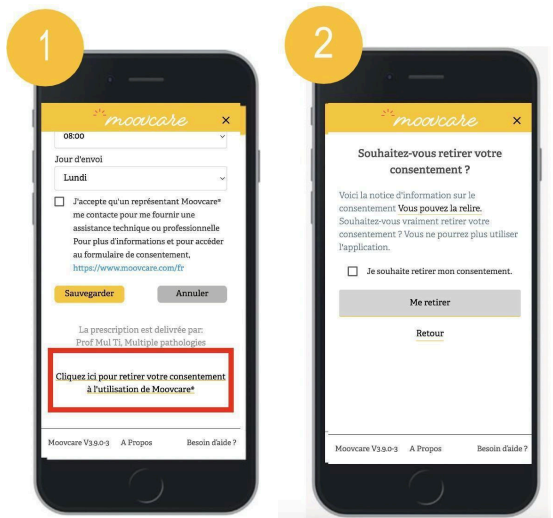

2.2.6. Documentation/aide

In the footer of the questionnaires, you will find information that you may need as part of your follow-up with Moovcare®.



2.2.7 Ask for help from a relative or caregiver

You can ask a relative or caregiver to fill out the questionnaire for you if needed. Please provide your doctor's contact information when you register.

The relative will not have separate access to the questionnaire.

Your consent to be contacted by a Moovcare representative for technical and professional support can be managed in the "Settings" section. By default, this consent is not selected, and the technical team does not have access to your contact information. You can enable or disable this option at any time.

3. Who will be eligible for Moovcare? Toxicity?

- Patients aged over 18 years.
- Patients with metastatic cancer undergoing treatment with chemotherapy, targeted therapies and/or immunotherapy

4. Warnings

- The effectiveness of Moovcare® Toxicity depends on your regular responses to the symptom questionnaire.
- If an anomaly is detected, an alert will be sent to your referring physician.
- If you have not completed the questionnaire, you will receive reminders until you do.
- If you are unable to complete the questionnaire on your own, a relative or caregiver can help you and submit your questionnaire, indicating the symptoms you are reporting.
- We recommend that the relative who will be assisting you complete the questionnaire.

be by your side to ensure that the answers are accurate.

- If you or your relative are unable to complete the questionnaire, please contact your referring physician.
- Moovcare® Toxicity does not replace a diagnosis made by a healthcare professional.
- If you are using other medical devices as part of your medical care, please inform your doctor before using Moovcare® Toxicity
- If new symptoms appear between two questionnaires, fill out a new questionnaire or contact your referring physician.
- If you no longer wish to use Moovcare® Toxicity, please inform your doctor.
- If you do not receive your questionnaires by email, please check your junk mail or spam folder.
- If you encounter any other difficulties not covered in this guide, please contact customer support by email at support@moovcare.com to inform us.
- Users are expressly informed that the use of Moovcare® Toxicity in a potentially dangerous environment is associated with IT security risks that cannot be fully addressed by the application manufacturer (e.g., if you save your login/passwords on a public computer).
- Do not use any automatic translators in your browser, as this may interfere with the wording. Moovcare® Toxicity supports four languages: French, English, German, and Spanish. If you wish to change the language, please do so through your account settings.
- In case of a serious incident, contact the support team immediately by email at support@moovcare.com as well as the competent authority in the country where you are located. It is also recommended to inform the agent by email at the following address: contact@kapsikum.com

A serious incident is defined as follows: any incident that directly or indirectly, likely to have caused or likely to cause:

the death of a patient, user, or any other person; a serious deterioration, temporary or permanent, in the health of a patient, user, or any other person; a serious threat to public health

6. Personal data

Using the Moovcare® Toxicity app requires you to enter personal data. You will need to provide personal information, including your email address and mobile phone number, to identify yourself digitally within Moovcare®. Your healthcare center has access to this administrative data to

facilitate communication.

Using the Moovcare® Toxicity app also requires your consent to the processing of your health data. This data is aggregated within Moovcare® Toxicity and is used to ensure you receive a questionnaire tailored to your postoperative follow-up over a predefined period (approximately 30 days) following your surgery. You will also provide information about your symptoms each time you complete the questionnaire.

This device is available in different versions, depending on the country in which it is used. A functional cookie allows you to be automatically redirected to the corresponding country after your first connection using the same browser and computer. A cookie is a text file that the server stores on your device. Your personal data collected in Moovcare® Toxicity is stored and secured with a certified health data hosting provider in France. You will find more information below. The Moovcare® Toxicity application is a medical device currently under investigation, and your data will be archived in accordance with applicable regulations.

The data collected is for study, research and development, or statistical purposes, for the needs of the manufacturer or its partners, or at the request of national or international health authorities. All data will be anonymized before any use.

Technical support will ask for your consent before answering your questions. You will also need to consent to the recording of the phone call. This data is stored on our provider's secure servers, which may be located outside of Europe. Our provider is committed to ensuring a level of security comparable to that required in Europe, through the use of binding contractual terms.

To facilitate your registration and monitoring with Moovcare® Toxicity, you also have the option of using Moovcare® Toxicity's professional assistance. You may have already consented to this during your registration with your doctor, or you can consent from your online account. By doing so, you authorize a nursing staff member at Sivan to access your personal data (name, surname, email address, phone number, status in Moovcare® Toxicity). You can withdraw your consent to this service at any time through the application.

In accordance with EU Regulation 2016/679 on the protection of natural persons with regard to the processing of personal data and on the free movement of such data (GDPR), you have the following rights concerning your personal data: right of access, right to rectification, right to erasure, right to restriction of processing, right to object to processing, right to data portability.

If you wish to exercise any of these rights, please contact the promoter's data protection officer. You can also contact the data protection officer (DPO) at Sivan.

to the following email address: dpo@sivan-innovation.com, by specifying your name, surname, email address, the name of your care center and the nature of your request.

For data portability purposes, you have the option to request direct access to your data in JSON file format. The file content does not require any specific data format. Your technical support data is transferred to Israel, to the application manufacturer. The European Union recognizes that Israel provides an adequate level of security for the protection of personal data.

We strive to comply with the GDPR to ensure the security of your personal data and protect your rights. However, if you believe there has been a breach of this regulation, you have the right to lodge a complaint with the National Commission for Information Technology and Civil Liberties (CNIL). (www.CNIL.fr) .

7. Symbols

Symbols used:

	Symbol indicating the manufacturer of the medical device		Warning symbol
	Symbol indicating the serial number with the software version		Symbol of Medical device
	Symbol indicating the unique identification number of the medical device	 2797	CE marking symbol, followed by the notified body number
	Symbol indicating consultation of the user manual		Symbol of the authorized representative in the European Union
	Website address		Indicates that an alert has been issued.
	Indicates that a photo can be viewed (png or jpeg)		Importer's symbol

8. Date of manufacture

Date of manufacture: see the "About" section at the bottom of the questionnaire pages. Date of revision: May 2026.

Publication date of the Moovcare® Toxicity information leaflet from Moovcare 1.0.1

9. Legal notice

Moovcare® is a medical device published by SIVAN INNOVATION Ltd, with a capital of 100,000 shekels, whose registered office is located at Hatidhar 5, Raanana, Israel, 51-509695-6.

Tel.: +972 2-567-7808 - Email: courrier@sivan-innovation.com

Publishing Director & Editorial Manager: Mr. Daniel Israel, President

SIVAN Innovation is represented by its agent Kapsikum, whose registered office is located at 4 rue René Razel 91400, Saclay, France

Email : contact@kapsikum.com

The Moovcare® lung application is hosted by SAS AVENIR TELEMATIQUE - ATE, with a share capital of €60,000, whose registered office is located at 21 Avenue de la Créativité, 59650 Villeneuve d'Ascq, and registered with the RCS of Lille MÉTROPOLE under number 347 607 764.

Tel.: (+33) 9 78 07 20 78 - Email: contact@ate.info

Legal representative: Mr. Maxence ROUSSEAU, President

Moovcare® lung, class IIa medical device <https://www.moovcare.com/>



MOOV-TOX_PATIENT_IFU_ENGLISH_05_26