




PATIENT AND CAREGIVER USER GUIDE

The Moovcare® Lung app is a medical device. In case of emergency, please contact your doctor or the emergency room.

Version 3.9.1

Notice publication: May 2026

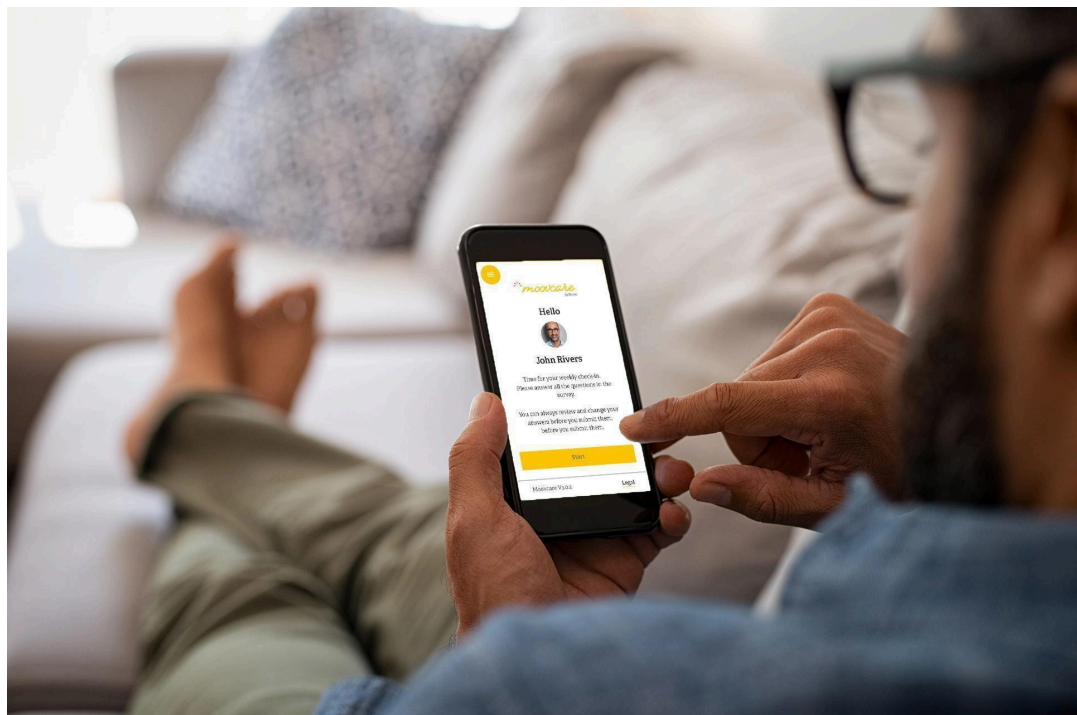

	Manufacturer:	SIVAN INNOVATION Ltd. Hatidhar 5 4366507 London, Israel
EU REP	Representative:	KAPSIKUM 4 rue René Razel 91400, Saclay, France
	Importer:	Sivan France 163 Quai Aulagnier F-92600 Asnières-sur-Seine, France

***Please read this leaflet carefully before using
MOOV CARE®Lung.***

- We advise you to keep this guide in case you need to review the web application's usage process. If you wish to access previous usage instructions or receive a printed copy, please send your request to:**contact@moovcare.com**.
- For any questions, advice or information, please contact your doctor.
- This medical device has been prescribed specifically for you. Do not share your access codes with anyone else, even if their symptoms are similar to yours, as this could compromise the proper functioning of this device for your medical monitoring.
- Technical support contact details:
support@moovcare.com:Response within 24 hours.

Service d'assistance - patients : 09 72 66 22 11 (free call)

Support hours: Monday to Friday, 9:00 AM to 12:00 PM and 2:00 PM to 5:00 PM
(except public holidays)



Sommaire

1. Operating principle.....	4
1.1. In practice.....	4
1.2. Professional healthcare users of Moovcare® Lung.....	4
2. Therapeutic indication, purpose and benefits.....	5
3. Instructions for use.....	5
3.1. Before we begin.....	5
3.2. Description of patient characteristics.....	7
3.2.1. Registration to benefit from the Moovcare® lung application.....	7
3.2.1.1. Step 1: Monitoring with the Moovcare® lung app.....	7
3.2.1.2. Step 2: Eligibility Criteria.....	7
3.2.1.3. Step 3: Activate your account.....	7
3.2.2. Your weekly questionnaire.....	12
3.2.3. How to change your settings?.....	14
3.2.4. View data history.....	16
3.2.5. Do you wish to stop the Moovcare® lung monitoring?.....	16
3.2.6. Documentation/aide.....	16
3.2.7. Ask for help from a relative or caregiver.....	16
4. Manage your prescriptions.....	17
5. Satisfaction Survey.....	18
6. Who can benefit from Moovcare®?.....	18
7. Warnings and contraindications.....	18
8. Personal Data.....	20
9. Symbols.....	22
10. Date of manufacture/CE marking.....	22
11. Legal notices.....	23


1. Operating principle

Moovcare® Lung is a web application that allows you to report your symptoms to your healthcare team each week, enabling them to intervene quickly if a problem arises. Moovcare® Lung uses the highest level of security to guarantee the confidentiality of your health data.

1.1. In practice

Your referring physician will ask you some questions before prescribing the Moovcare® Lung application. Once a week, you will receive an email with a link to a simple questionnaire for you. This allows you to report your symptoms on your phone, tablet, or computer. If Moovcare® Lung detects an abnormality, your healthcare team will be notified.

Une nouvelle approche dans le suivi du cancer du poumon

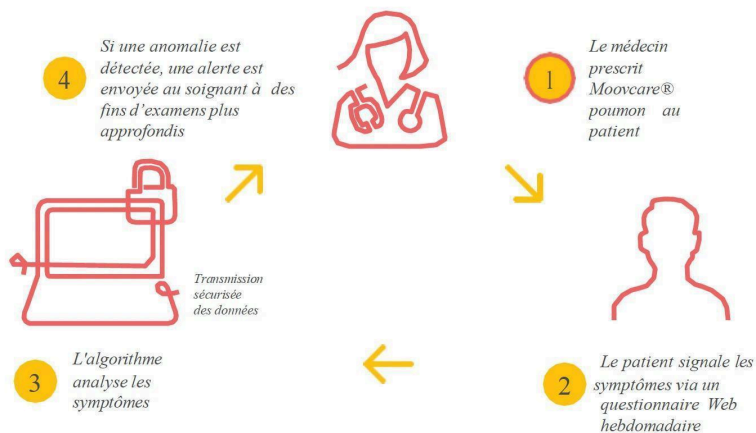

1.2. Professional users of Moovcare® Lung

Referring physician

Your doctor prescribes the Moovcare® Lung monitoring app. All your responses are available on the doctor's dashboard to continuously monitor your health. If an abnormality is detected, an alert is sent.

The other doctors at the care center

In case of absence, the referring physician may designate a secondary physician to monitor your symptoms.

The medical staff and the doctor's associates

Staff members designated by doctors may access your data to facilitate your follow-up.

2. Therapeutic indication, purpose and benefits

The Moovcare® lung application is designed to remotely monitor patients with lung cancer in order to detect any relapse or complication early.

Eligible patients:

- Patients with lung cancer aged over 16 years.
- Non-progressive cancer after evaluation of the last treatment.
- Having internet access and an active mobile number or being able to be helped by relatives to complete the weekly questionnaire.

Moovcare® lung can be used by patients undergoing treatment.

Moovcare® lung has undergone a rigorous clinical trial program. A phase III study¹ compared Moovcare® lung *versus* standard monitoring.

¹F. Denis, Lethrosne C, Pourel N, Molinier O, Pointreau Y, Domont J, et al. Randomized trial comparing a Web-mediated follow-up with routine surveillance in lung cancer patients. J Natl Cancer Inst. 2017 Sep 1;109(9). doi: 10.1093/jnci/djx029.

The study demonstrated the following benefits:

- Detection of relapses or complications.
- An increase in overall survival of more than 7.6 months.
- An improvement in the patient's quality of life.
- An improvement in the patient's overall health.

3. Instructions for use

3.1. Before we begin

Moovcare® lung is a medical monitoring device available only on prescription from a healthcare professional.

The app does not replace a doctor's diagnosis but functions as a digital support tool for remotely monitoring the symptoms of lung cancer patients. Moovcare® allows you to record your symptoms each week, and this information will be immediately transmitted to your doctor.

Note : At the end of your questionnaire, you have a free text field to allow you to provide further medical information or report new symptoms.

This free text field is reserved solely for communicating medical information (new symptoms) or for alerting the practitioner in case of an error in a previous questionnaire.

Personalized tracking

Moovcare® lung allows for personalized monitoring and ongoing communication with your referring physician through a weekly questionnaire. Moovcare's accuracy® depends on the weekly completion of the questionnaire.

No installation required

Moovcare® lung is a web application; no download or installation is required. Internet access and an email address are mandatory. We recommend using a Gmail account to receive emails and notifications.

Moovcare® lung V.3.9.1 is compatible with the following versions of search engines:

- Chrome 141
- Firefox 141
- Internet Explorer 11
- Safari 18
- Edge 144

And also with system operators:

- Android 16
- iOS 26
- OSX 26
- Windows 11

Data security

Moovcare® lung uses the highest level of security to guarantee the confidentiality of your health data.

All databases linked to the system comply with national data protection regulations. The data centers are located in France within a company certified to ISO 27001:2013 for hosting health data and accredited by the French Ministry for storing health data.

Weekly questionnaire

Once a week, you will receive an email with a link to a simple questionnaire to allow you to report your symptoms on your phone, tablet or computer.

You can also use Moovcare® lung to report symptoms that may appear at any time between two questionnaires.

What kind of questions will I be asked?

Moovcare® Lung will ask you to report any typical symptoms you may be experiencing and indicate their severity, if applicable. This may be a yes/no question, a symptom rating from minor to major, and a request to enter your weight and temperature. Your referring physician will be notified if the algorithm detects any abnormalities. In case of acute chest pain, with or without shortness of breath (dyspnea), please contact a doctor or the emergency department immediately.

Note :

- *Your temperature must be taken with the same thermometer. The weight you indicate in each questionnaire must be measured with the same scale/bathroom scale, without clothing and in the morning on an empty stomach.*
- *On the last screen of the questionnaire, you will need to validate your answers and click on*

"send" to transmit them to the Moovcard® lung system.

3.2. Description of patient characteristics

3.2.1. Registration to benefit from the Moovcare® lung application

3.2.1.1. Step 1: Monitoring with the Moovcare® lung app

Moovcare® lung is a medical device available only on prescription from a healthcare professional. Ask your doctor if Moovcare® lung is available at your healthcare facility and if you meet the eligibility criteria.

3.2.1.2. Step 2: Eligibility Criteria

To determine your eligibility for Moovcare® lung monitoring, your doctor must ensure that you meet the following conditions:

- You are over 18 years old.
- You are not symptomatic. Your doctor will ask you 5 questions to determine if you are.

Note : When your doctor enrolls you in the Moovcare® lung system, he/she records your medical information about your current condition.

You can choose a family member or trusted person (such as a home caregiver) to help you complete the Moovcare® Lung Questionnaire at any time. This person will also receive a reminder if you miss a questionnaire.

If you deem it necessary, please inform your referring physician.

3.2.1.3. Step 3: Activate your account

Once registered on Moovcare® lung by your referring physician (or an accredited member of the care team), **You will receive an email to activate your account..**

- If you don't see an email in your inbox, check your junk mail or spam folder. Once you find the email, mark it as "legitimate" so that future emails will arrive in your inbox.
- If you cannot find the activation email in your spam folder, the email address you entered during registration is likely incorrect. Please contact your healthcare center for assistance.

And then what?

1



You will receive an email to confirm the activation of your account, entitled "Welcome to Moovcare® lung".

This email will give you access to the Moovcare® lung system documentation and allow you to start filling out your first questionnaire.

Note : You will be asked for your password each time you wish to log in to the application. You can save it to your device so that your browser recognizes it

We recommend that you set a new password if you believe your account or password has been compromised.

2



You will be asked to confirm your email address. Click Confirm to open the Moovcare® lung app; you will then be asked to create a password of at least 12 characters, including numbers and letters. Special characters (@, #, %, etc.) can be used. First or last names cannot be used. The last 3 passwords cannot be reused.

Once the password has been created, click on **Set the password**.

When creating your password, you will be informed of the strength of the password you choose.

3



The login screen features a yellow header with the Moovcare logo. Below the header, the text "Bienvenue !" is displayed. Underneath, the instruction "Veuillez vous identifier" is shown. There are two input fields: the first for the email address, which contains "moovcare@mail.com", and the second for the password, which contains eight asterisks. A yellow button labeled "M'identifier" is positioned below the password field. A link "Mot de passe oublié ?" is located below the button. At the bottom of the screen, there is a footer with three links: "Moovcare V3.7.", "A Propos", and "Besoin d'aide ?".

The login screen will appear. Enter your password and click on ***M'identifier***.

4



The consent screen features a yellow header with the Moovcare logo. Below the header, the text "Consentement :" is displayed. Underneath, there is a paragraph of text explaining the collection and use of health data. Below the text, there is a checkbox labeled "Je donne mon consentement". A grey button labeled "Poursuivre" is positioned below the checkbox. At the bottom of the screen, there is a footer with three links: "Moovcare V3.7.", "A Propos", and "Besoin d'aide ?".

The consent screen will open. Read it carefully; it concerns your agreement on the collection and use of your health data in Moovcare® lung. Check the box ***I give my consent*** then click on ***Pursue***.

5

Next, the Terms of Use will appear. Read the **Conditions General Terms of Use**, check the box *I agree* then click on **Continue**.

6

Provide information your personal information, then click on **Confirm**.

Note : If there is an error in your phone number (you can check the one used to send the SMS on this same screen), click on **Change phone number** to correct it and receive a new code.

7



You will then receive a text message on your phone with a code to identify yourself. Go back to the Moovcare® lung app and enter the authentication code, then click on **Confirm**.

3.2.2. Your weekly questionnaire



Your first weekly questionnaire

Once your account is activated, click on **To start** in order to complete your first questionnaire.

Each week, you will receive an email with a link to your Moovcare account. To complete your weekly questionnaire, click on **To start** and send it to your doctor.



Answer your questionnaire

By clicking on **To start**, You will be directed to the first question. You will be asked to report any characteristic symptoms you may have and to indicate its severity where applicable.

3

Did you forget or delay in answering the questionnaire?

If you do not complete the questionnaire within 24 hours of receiving the link, you and your designated contact person (if applicable) will receive a reminder after 3 days. After 10 days, your doctor will also receive a notification, as will you and your contacts.

Once you have submitted your questionnaire, please check that you have received the email confirmation. If not, we recommend that you submit a new questionnaire.


Note :

- Your weight must be measured with the same scale/bathroom scale, in the morning on an empty stomach and without clothes.
- Your temperature should be taken every week with the same thermometer. If your temperature exceeds 38.2°C, click on **Yes** and a window will open allowing you to enter your temperature.
- At the end of the questionnaire, you will find a free text field. This is strictly reserved for transmitting medical information such as a change in your health status or new symptoms not specified in the questionnaire.
Check your answers, then click on **Send**. You will no longer be able to change your answers. After this screen, you can close the page and you will receive a confirmation email shortly.
- You can also use Moovcare® lung to report symptoms that may appear at any time between questionnaires using this address: <https://patient.moovcare.com>


How to Request a New Questionnaire

? If you wish to submit a new questionnaire, you can do so by logging in to the website: <https://patient.moovcare.com> **with**

3.2.3. How do I change my settings?

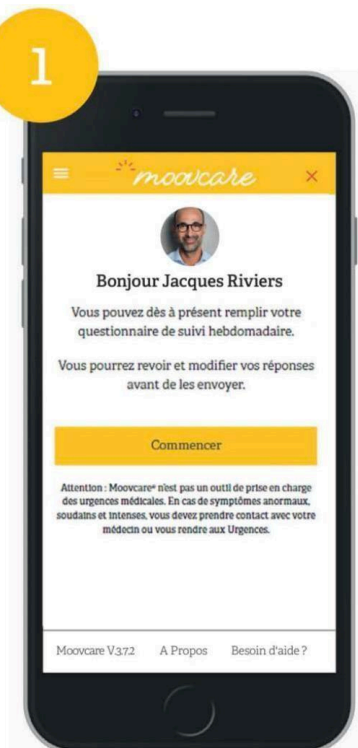

You can directly manage and modify certain data concerning you. On the homepage, click on the menu in **top left**.

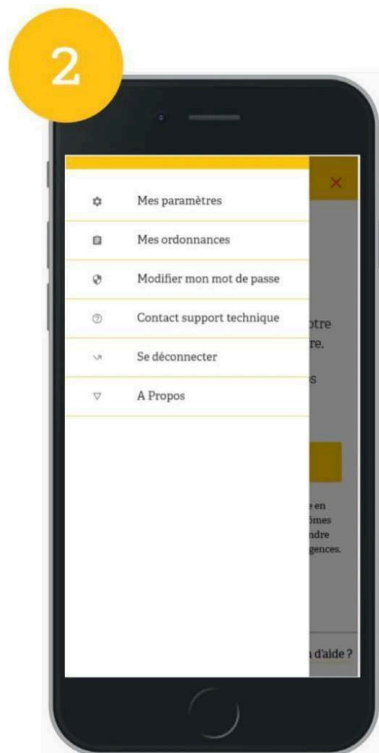

Select **Change my password**.

3

Modifiez le mot de passe

Vous pouvez changer votre mot de passe ici

Modifier

[Annuler](#)

Le mot de passe doit contenir au moins 12 caractères en incluant chiffres et lettres.

Moovcare V3.7.2 A Propos Besoin d'aide ?

4

Changer la photo de profil

Email
moovcare@moovcare.moovcare

Téléphone
+(33) 6 65 43 21 98

Prénom
Test

Nom
Test

Langue
Français

Tous les champs sont requis

Enregistrer

Moovcare V3.7.2 A Propos Besoin d'aide ?

Please enter your new password, which must be at least 12 characters long and include numbers and letters. Special characters are permitted. To reset your password, we require two-factor authentication using a code sent via SMS to your mobile phone, which you must enter in the app. Please note that your new password must be different from your previous three passwords.

In **My settings**, you can change the **language of the questionnaire** as well as **the time and day your questionnaire was sent**.

Note: when you want to make a change to your settings, an authentication code will be sent to you via your phone.

3.2.4. View data history

You have the right to access your medical data. Your referring physician can export your questionnaire history if necessary.

3.2.5. Do you wish to stop the Moovcare® lung monitoring?


If you no longer wish to be monitored by Moovcare® lung, you can withdraw your consent.

On the Welcome page, select **My settings** in the menu at the top left.

At the bottom of the list, you will see **Withdraw my consent for Moovcare®**.

By clicking on this option, a screen will appear to confirm that you really want to stop the Moovcare® lung monitoring.

If you wish to resume your Moovcare® lung care, please contact your referring physician. They will send you an email allowing you to log back in using your email address and the same password you used previously. You will then need to provide your consent again.

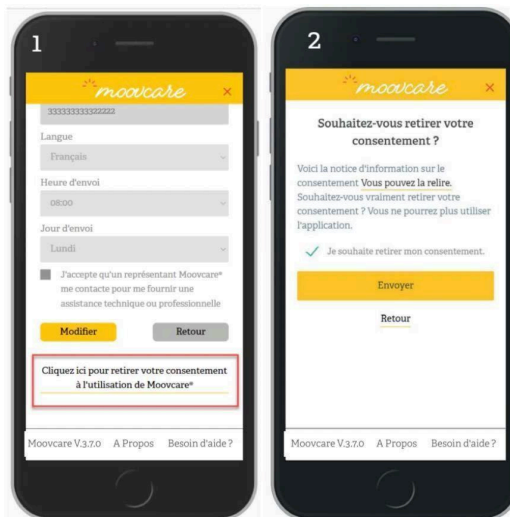

3.2.6. Documentation/aide

In the footer of the questionnaires, you will find information that you may need as part of your follow-up with Moovcare® lung.

3.2.7. Ask for help from a relative or caregiver

You can ask a relative or caregiver to fill out the questionnaire for you if necessary. Please provide this contact information to your doctor when you register. The relative will not have separate access to the questionnaire.

If you do not complete the questionnaire every week, your loved one will receive a reminder.

Your consent to be contacted by a Moovcare representative for technical and professional support can be managed in the “Settings” section. By default, this consent is not selected, and the technical team does not have access to your contact information. You can enable or disable this option at any time.

Note: we advise you not to send the link to a relative, but you can fill out the questionnaire with them.




4. Manage your prescriptions

Moovcare® lung is a reimbursed product in France, which means you can only obtain it with a prescription from your referring physician. This prescription must be renewed every 6 months.

Your prescription must always be valid. Each prescription can be classified into one of 3 status categories:


1. **Valid order**– the date of the prescription is still valid for at least one month
2. **Prescription about to expire**– the prescription will expire in 30 days or less
3. **Prescription expired**- the prescription has expired, you cannot use Moovcare® lung until updated with a valid prescription.

When it's time to renew your prescription, we'll remind you, but if you'd like to do it before the reminder, please follow these steps:

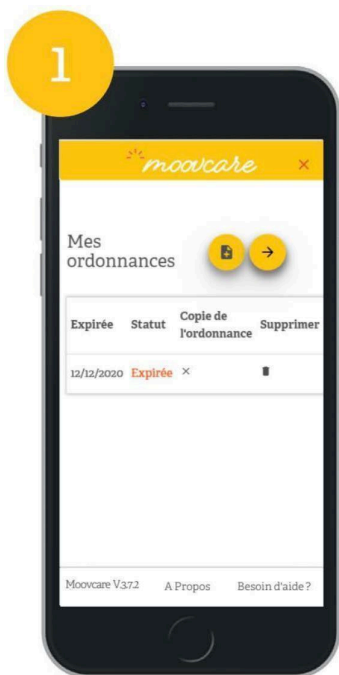

Open the left menu and select **My prescriptions**. Your current and previous prescriptions will be displayed.

If you want to add a new prescription, click on the icon in the top right corner. **Add a prescription.**

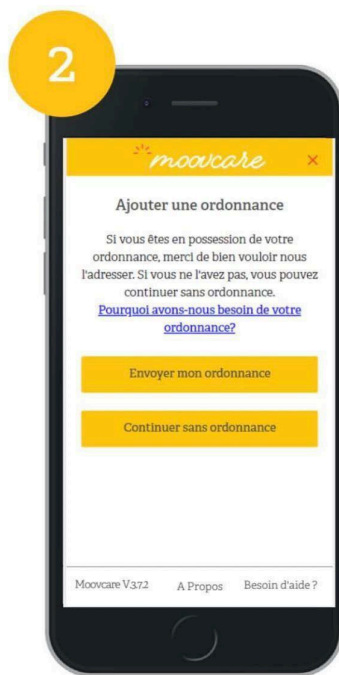

You will need to select the date the prescription was issued and upload an image of the prescription. You can take a photo with your phone and upload it in any image format (jpg, png or gif).

Next, click on **Confirm** and the order will be updated.

5. Satisfaction survey

At Sivan Innovation, we are committed to continuously improving Moovcare® for your well-being and convenience. To further enhance Moovcare®, SIVAN conducts a user satisfaction survey. This optional survey is managed independently of the application. SIVAN Innovation and SIVAN France are the sole recipients of this information, which will be anonymized and used for statistical purposes in the form of internal reports. You will be invited to participate in this optional survey every two months.

6. Who can benefit from Moovcare®?

- Patients with lung cancer aged over 16 years.
- Moovcare® lung is not suitable for patients with significant cognitive problems; However, they can get help from a relative during the follow-up. This relative must be registered in The application will be used, and information will be sent via email. A patient with a cognitive problem is a patient who lacks the ability to complete the questionnaire correctly.
- Eligibility: Your referring physician will ask you 5 questions to verify if you are eligible for follow-up by Moovcare® poumon.

7. Warnings and contraindications

The effectiveness of Moovcare® Lung depends on your regular responses to the weekly questionnaire. If an abnormality is detected, an alert will be sent to your referring physician.

If you have not yet completed the questionnaire, you will receive reminders until you do.

If you are unable to complete the questionnaire on your own, a family member or caregiver can help you and submit it, indicating the symptoms you are reporting. We recommend that the person assisting you be present to ensure the accuracy of your answers.

If you or your loved one are unable to complete the questionnaire, please contact your primary care physician.

Moovcare® lung is not a substitute for a diagnosis made by a healthcare professional.

If you use other medical devices as part of your medical care or if you are taking medication, please inform your doctor before using Moovcare® Lung. The effectiveness of Moovcare® Lung has been demonstrated provided the questionnaire is completed weekly. If new symptoms appear between questionnaires, please contact your doctor.

referring physician.

If you no longer wish to use Moovcare® lung, please inform your doctor.

If you do not receive your questionnaires by email, please check your junk mail or spam folder.

If you encounter any other difficulties not covered in this guide, please contact customer support by email at support@moovcare.com to inform us. Or by telephone at...

Users are expressly informed that using Moovcare® Lung in a potentially hazardous environment is associated with security risks that cannot be fully mitigated by the digital health application's manufacturer. (Example: saving login/password on a public computer). Access codes (passwords) are personal and must not be shared.

Do not use any automatic translators in your browser, as this may distort the wording. Moovcare supports three languages: French, English, and Italian. If you wish to change the language, please do so through your account settings.

There are no warnings associated with using the application.

Residual risk

The expected risks for this application are minimal but may nevertheless involve psychological harm such as emotional distress or embarrassment related to the questionnaire questions and breach of confidentiality.

The questionnaire questions will be used in the clinical trial, and any signs of harm will be assessed. In general, it is unlikely that the application will cause harm. However, if a patient experiences emotional distress as a result of these questions, psychological counseling should be sought.

The risk of a breach of confidentiality will be minimized in several ways, as outlined below:

Moovcare® is a web-based algorithm in which patients enter symptom data via a password-protected digital platform. Registered participants receive a unique password to log in to the site, ensuring that only they, a healthcare provider with whom they share the password, and their care team can access their PRO data. For the study, data, including outcome questionnaires and graph abstraction forms, will be collected via a web platform by the secure survey administration.

For the study – Strict confidentiality will be maintained. Paper-based research data will be minimal and stored in locked, secure areas accessible only by the study team. Research data will be kept in separate tables and identified by an ID number only when possible. A master list linking names and ID numbers will be kept in a separate, secure location. Only approved members of the research team will have access to the secure files and will be informed about the protection of patients' rights to confidentiality. At the end of the study, once the database has been declared complete and accurate, it will be locked.

Data transmitted between the server and end users is encrypted using SSL, and all databases are encrypted. Research team members will be responsible for maintaining the confidentiality of all participant data and patient participation.

Contraindications:

- MOOV CARE® Lung is not suitable for persons under 18 years of age.
- MOOV CARE® Lung can be used by pregnant women. However, it is necessary to take into account the evolution of symptoms related to this condition.
- MOOV CARE® Lung is not suitable for people with significant cognitive impairment. However, these individuals can be assisted by a family member or friend during their follow-up. This person must be designated by the patient at registration to be enrolled and receive information by email. The patient has the option to add a family member or trusted individual at any time during their follow-up.
- MOOV CARE® Lung is not suitable for patients with severe symptoms at the start of their lung cancer monitoring. A series of 5 questions to be asked and completed during patient enrollment will help determine this condition.
- In case of a serious incident, contact the support team immediately by email atsupport@moovcare.com as well as the competent authority in the country where you are located. It is also recommended to inform the agent by email at the following address: contact@kapsikum.com

A serious incident is defined as follows: any incident that has directly or indirectly resulted in, is likely to have resulted in, or is likely to result in:

the death of a patient, user or any other person;

a serious deterioration, temporary or permanent, in the health of a patient, user or any other person; a serious threat to public health

8. Personal data

Using the Moovcare® Lung app requires you to enter personal data. Your doctor or the healthcare professional who handled your initial registration in the system will be responsible for collecting your data. You will need to provide personal information, including your address, landline number, mobile phone number, and social security number. Your healthcare center has access to this administrative data to facilitate communication. Using the Moovcare® Lung app also requires your consent to the processing of your health data. During your initial registration, your doctor will ask you some questions about your health to confirm your eligibility. You will also provide health data each time you complete the questionnaire.

SIVAN France will have access to the name, first name(s), status in Moovcare®, social security number, date of birth, and personal data present on the Moovcare® lung prescription of the insured person receiving reimbursement.

This feature is available in different versions, depending on the country in which it is used. A functional cookie allows you to be automatically redirected to the corresponding country after your first login using the same browser and computer. A cookie is a text file that the server stores on your device.

This data is then communicated to the health insurance provider and any intermediary providers in order to organize the financial transactions.

Your personal data is stored securely with a certified healthcare data hosting provider in France. You will find more information below. The Moovcare® lung application is a regulated medical device and, as such, all recorded data must be secured and archived for ten years.

SIVAN Innovation may use the data for studies, research and development, or statistical purposes, for its own needs or those of its partners, or at the request of national or international health authorities. All data will be anonymized before any use.

Technical support will ask for your consent before answering your questions. You will also need to consent to the recording of the phone call. This data is stored on our provider's secure servers, which may be located in the USA. Our provider is committed to ensuring a level of security comparable to that required in Europe, through the use of binding contractual terms.

To facilitate your Moovcare® registration and follow-up, you also have the option of using Moovcare®'s professional assistance. You may have already consented to this during your registration with your doctor, or you can consent from within your account. By doing so, you authorize a nursing staff member at Sivan to access your personal data (name, surname, email address, phone number, Moovcare® status). You can withdraw your consent to this service at any time through the application.

In accordance with EU Regulation 2016/679 on the protection of natural persons with regard to the processing of personal data and on the free movement of such data (GDPR), you have the following rights concerning your personal data: right of access, right to rectification, right to erasure, right to restriction of processing, right to object to processing, right to data portability.

If you wish to exercise any of these rights, please contact your referring physician. You can also contact the Data Protection Officer (DPO) at Sivan at the following email address: dpo@sivan-innovation.com by specifying your name, surname, email address, the name of your care center and the nature of your request.

For data portability purposes, you have the option to request direct access to your data in JSON file format. The file content does not have a specific data format requirement. You can

receive: questionnaire response history, administrative and medical data, email history, and alerts (symptoms triggered, date, time, and doctor who closed the alert).

We strive to comply with the GDPR to ensure the security and protection of your rights regarding your personal data. However, if you believe there has been a breach of this regulation, you have the right to lodge a complaint with the National Commission for Information Technology and Civil Liberties (CNIL). www.CNIL.fr

or to the data protection officer (DPO) of the data processing manager (the DPO of the care center or the application manufacturer) dpo@sivan-innovation.com).

If you had consented to participate in a survey conducted by SIVAN INNOVATION and SIVAN France and you no longer wish your data to be used, you can then contact our DPD to exercise your rights over this data (address above).

A survey will be sent every two months to gather your feedback on improvements following the use of Moovcare® Lung and additional information regarding the use of the application. We do not collect any personal data that could lead to your identification. Your consent is required for this survey.

Please note that if you do not respond to this survey, you will continue to receive it.

Some of your data (technical support, survey, or billing) is transferred to Israel, to the application developer. The European Union recognizes that Israel has an adequate level of security to protect personal data.

9. Symbols

Symbols used:

	Symbol indicating the manufacturer of the medical device		Warning symbol
	Symbol indicating the serial number with the software version		Medical Device Symbol
	Symbol indicating the unique identification number of the medical device		CE marking symbol, followed by the notified body number
	Symbol indicating consultation of the user manual		Symbol of the authorized representative in the European Union
	Website address		Indicates that an alert has been issued.
	Indicates that a photo can be viewed (png or jpeg)		Importer's symbol

10. Date of manufacture/CE marking

Holder of the CE marking:SIVAN INNOVATION LTD

Obtaining CE marking: 2017-07

Date of manufacture:See the "About" section at the bottom of the page.

Date of revision:Date of publication of the Moovcare® lung leaflet from Version 3.9.1
May 2026

11. Legal notice

Moovcare® is a medical device published by SIVAN INNOVATION Ltd, with a capital of 100,000 shekels, whose registered office is located at Hatidhar 5, Raanana, Israel, 51-509695-6.

Tel.: +972 2-567-7808 - Email: courrier@sivan-innovation.com

Publishing Director & Editorial Manager: Mr. Daniel Israel, President

SIVAN Innovation is represented by its agent Kapsikum, whose registered office is located at 4 rue René Razel 91400, Saclay, France

Email : contact@kapsikum.com

The Moovcare® lung application is hosted by SAS AVENIR TELEMATIQUE - ATE, with a share capital of €60,000, whose registered office is located at 21 Avenue de la Créativité, 59650 Villeneuve d'Ascq, and registered with the RCS of Lille MÉTROPOLE under number 347 607 764.

Tel.: (+33) 9 78 07 20 78 - Email: contact@ate.info

Legal representative: Mr. Maxence ROUSSEAU, President

Moovcare® lung, class IIa medical device <https://www.moovcare.com/>

